

## 2D Measurements vs. 3D Measurements for Total Shoulder Replacement

Gilles Walch MD Jean Chaoui PhD

This white paper is a publication of Wright Medical Group N.V.  ${\scriptstyle @2016}$  Wright Medical Group N.V. All Rights Reserved.



Error Sources in 2D Glenoid Measurements	Why are 3D Glenoid Measurements More Accurate?
Patient positioning in the CT machine introduces errors in the three anatomical planes (coronal, sagittal and axial)	Measurements in 3D are completely independent of patient positioning. In others words, a patient positioned in three different orientations will all produce the same glenoid version and inclination measurements.
Version and inclination 2D measurements change through slices progressing further from the midline superior-to-inferior and anterior-to-posterior	Version and inclination 3D measurements are calculated using all of the points on the face of glenoid to form a best-fit sphere.
<ul> <li>The 2D humeral head subluxation is based on:</li> <li>1) Both the Friedman's line which is supposed to represent the scapular plane</li> <li>2) The assessment of the humeral head diameter only</li> </ul>	3D humeral head subluxation is based on both the scapular plane generated from the entire scapula, and the whole humeral head volume.

There are several advantages of 3D measurements, such as those calculated by BLUEPRINT<sup>™</sup> 3D Planning Software (Figure 1), to 2D measurements when performing a pre-operative planning for total shoulder arthroplasty (Figure 2). The aim of existing 3D methods is to provide more accurate and reliable data of native shoulder anatomy, such as glenoid version/inclination, glenoid wear pattern and humeral head subluxation.

Currently, numerous surgeons are using either 2D radiographs or 2D-CT slices as gold standards for planning the surgeries. Hoenecke et al.<sup>1</sup> have proved that "standard 2D-CT slices were not as accurate as 3D reconstructions for measurements of glenoid version and for locating the direction of maximum wear". The authors support the need for full 3D-CT reconstruction for preoperative planning in complex cases. Budge et al.<sup>2</sup> reported that 2D measures underestimate glenoid retroversion compared to 3D measures. Axial 2D images were 5° to 15° different than the 3D measures in 47% of the measurements.



**Figure 1.** "Advanced measures" screen, available in BLUEPRINT" 3D Planning Software, providing specific measurements of the native shoulder anatomy.



**Figure 2.** Main window of BLUEPRINT" 3D Planning Software for selecting the suitable position of the glenoid implant in accordance to the version, the inclination, the reaming depth and the seating.

This white paper is a publication of Wright Medical Group N.V. ©2016 Wright Medical Group N.V. All Rights Reserved.

In addition, Terrier et al.<sup>3</sup> showed that the mean version measured in 2D to obtain the classification of Walch was 9°. It was significantly lower in 2D than in 3D for A1 and B2. The version was under-evaluated in 2D by more than 5° and 10° in 34% and 13% of cases respectively.

The glenoid inclination is also an important parameter highlighted by 3D method emergence. Daggett et al.<sup>4</sup> concluded that the ß-angle (defined by Maurer et al.<sup>5</sup>) measured with 2D CT scan formatted in the scapular plane using BLUEPRINT<sup>™</sup> 3D Planning Software, was the most accurate method for measuring glenoid inclination. This technique was compared to two others methods using 2D radiographs and unformatted 2D CT scan. The authors confirm that "the 3D software provides the closest depiction of scapular anatomy."

The reliability and reproducibility of BLUEPRINT<sup>®</sup> 3D Planning Software has also been demonstrated by Moineau et al.<sup>6</sup> when calculating several parameters of arthritic glenoid cavities. The authors declared that "these 3D measurements are advantageous because they are free of problems related to patient positioning in the CT scanner and to the choice of slices, which limits the accuracy of measurements made on slices from 2D CT scans."

A suitable pre-operative planning also involves the humeral component, which is strongly related to the glenoid and rotator cu wears. Terrier et al.<sup>7</sup> have shown that 3D measurement of scapulohumeral subluxation should be preferred to the usual 2D measurement of glenohumeral subluxation. Jacxsens et al.<sup>8</sup> compared humeral subluxation on 2D and 3D imaging, and determined that **2D measurements underestimated posterior subluxation compared to 3D measurements.**  These studies illustrate that 3D measurements allow surgeons to get a more accurate representation of the real patient shoulder. In addition, future technological improvements will likely o er the possibility to automatically estimate bone density distribution of the glenoid vault and the range of motion after positioning a total shoulder implant, options which are not available with standard 2D methods.

This white paper is a publication of Wright Medical Group N.V. ©2016 Wright Medical Group N.V. All Rights Reserved.

## References

- 1. Hoenecke HR Jr, Hermida JC, Flores-Hernandez C, D'Lima DD. Accuracy of CTbased measurements of glenoid version for total shoulder arthroplasty. J Shoulder Elbow Surg. 2010 Mar;19(2):166-71.
- 2. Budge MD, Lewis GS, Schaefer E, Coquia S, Flemming DJ, Armstrong AD. Comparison of standard two-dimensional and threedimensional corrected glenoid version measurements. J Shoulder Elbow Surg. 2011 Jun;20(4):577-83
- Terrier A, Ston J, Larrea X, Farron A. Measurements of three-dimensional glenoid erosion when planning the prosthetic replacement of osteoarthritic shoulders. Bone Joint J. 2014 Apr;96-B(4):513-8.
- Daggett M, Werner B, Gauci MO, Chaoui J, Walch G. Comparison of glenoid inclination angle using different clinical imaging modalities. J Shoulder Elbow Surg. 2016 Feb;25(2):180-5.
- 5. Maurer A, Fucentese SF, Pfirrmann CW, Wirth SH, Djahangiri A, Jost B, et al. Assessment of glenoid inclination on routine clinical radiographs and computed tomography examinations of the shoulder. J Shoulder Elbow Surg 2012;21:1096-103.

- Moineau G, Levigne C, Boileau P, Young A, Walch G; French Society for Shoulder & Elbow (SOFEC). Three-dimensional measurement method of arthritic glenoid cavity morphology: feasibility and reproducibility. Orthop Traumatol Surg Res. 2012 Oct;98(6 Suppl):S139-45.
- 7. Terrier A, Ston J, Farron A. Importance of a three-dimensional measure of humeral head subluxation in osteoarthritic shoulders. J Shoulder Elbow Surg. 2015 Feb;24(2):295-301.
- 8. Jacxsens M, Van Tongel A, Willemot LB, Mueller AM, Valderrabano V, De Wilde L. Accuracy of the glenohumeral subluxation index in nonpathologic shoulders. J Shoulder Elbow Surg. 2015 Apr;24(4):541-6.



Proper surgical procedures and techniques are the responsibility of the medical professional. This material is furnished for information purposes only. Each surgeon must evaluate the appropriateness of the material based on his or her personal medical training and experience. Prior to use of any Tornier implant system, the surgeon should refer to the product package insert for complete warnings, precautions, indications, contraindications, and adverse effects. Package inserts are also available by contacting Wright. Contact information can be found in this document and the package insert.

<sup>¬°</sup> and <sup>®</sup> denote Trademarks and Registered Trademarks of Wright Medical Group N.V. or its affiliates. ©2016 Wright Medical Group N.V. or its affiliates. All Rights Reserved.