

HAELVOET S.A.R.L. **FRANCE** 12, AVENUE VAN PELT, 62300 LENS T +33 (0) 3 21 70 63 71 F + 33 (0) 3 21 76 62 67 INFO@HAELVOET.FR WWW.HAELVOET.FR

> HAELVOET S.R.L **ROMANIA** PARCUL INDUSTRIAL 1,455200 JIBOU, SALAJ T 0040 260 64 22 33 F 0040 260 64 22 55 INFO@HAELVOET.RO WWW.HAELVOET.RO

HOSPITAL - NURSING HOME FURNITURE


HAELVOET NV **BELGIUM** L. BEKAERTSTRAAT 8, 8770 INGELMUNSTER T +32 (0) 51 48 66 95 F +32 (0) 51 48 73 19 INFO@HAELVOET.BE WWW.HAELVOET.BE

HAELVOET SA **SWITZERLAND** RUE DU CRÊT-DU-LOCLE 4B, 2304 LA CHAUX-DE-FONDS T +41 (0) 79 700 86 46 F +41 (0) 21 560 46 46 INFO@HAELVOET.CH WWW.HAELVOET.CH



# Olympia Hospital



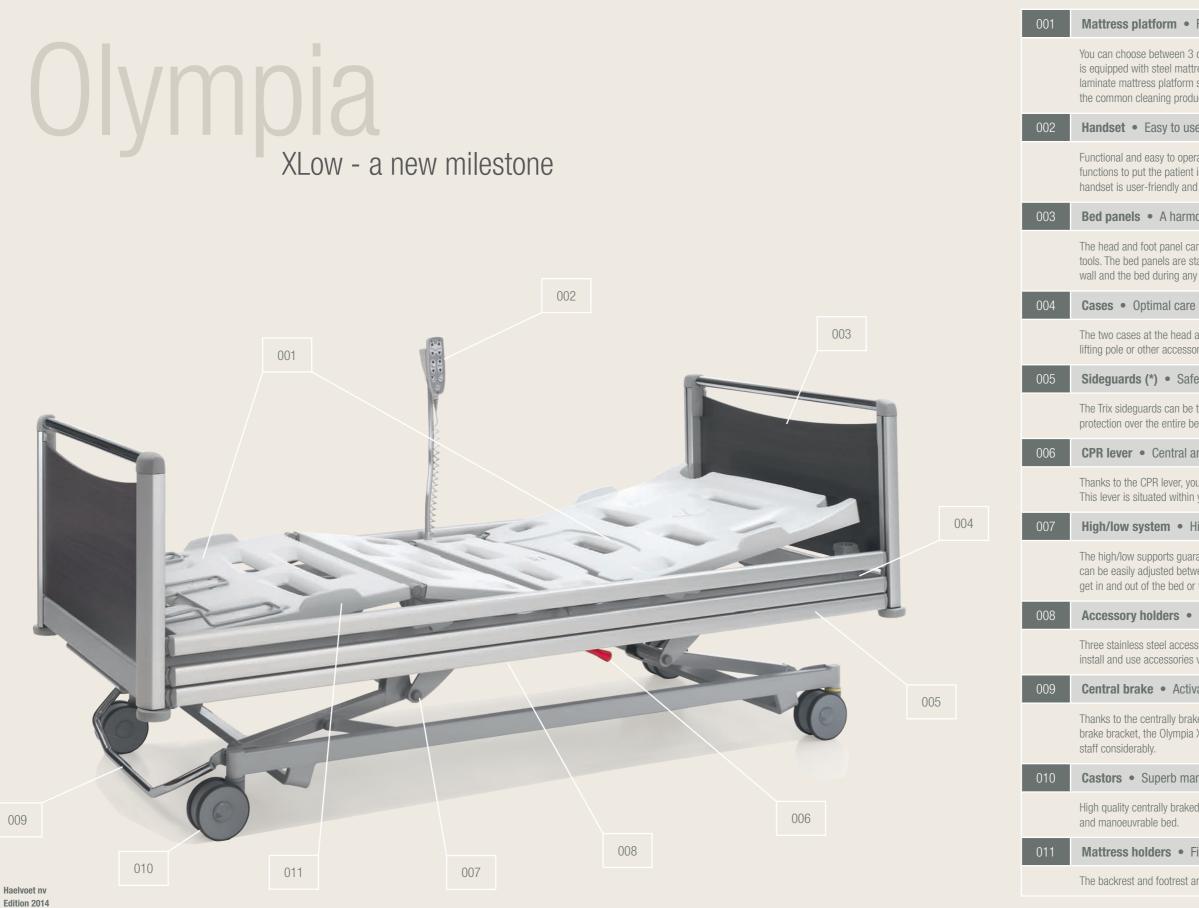

Haelvoet reserves the right to modify the product. Small colour deviations are possible.

#### **Mattress platform** • For a comfortable lying position

You can choose between 3 different mattress platforms. The standard version is equipped with steel mattress platform profiles. The optional synthetic or solid laminate mattress platform sections (\*) can be easily removed and withstand the common cleaning products.

#### Handset • Easy to use autonomously

Functional and easy to operate manually. The handset is equipped with all functions to put the patient in his desired seating, lying or reading position. The handset is user-friendly and equipped with clear and recognizable push buttons.

#### **Bed panels** • A harmonious duo

The head and foot panel can be easily removed or installed without needing any tools. The bed panels are standard equipped with bumpers that protect both the wall and the bed during any movement.

The two cases at the head and foot of the bed can be used to install an IV rod, a lifting pole or other accessories.

#### Sideguards (\*) • Safety has no price

The Trix sideguards can be telescopically extended and offer maximum patient protection over the entire bed length.

#### **CPR lever** • Central and within reach

Thanks to the CPR lever, you can manually put the backrest in a horizontal position. This lever is situated within your reach on both sides of the bed.

#### High/low system • Highly stable

The high/low supports guarantee an excellent bed stability at all heights. The bed can be easily adjusted between  $\pm$  25 cm and  $\pm$  80 cm, which makes it easier to get in and out of the bed or to treat a patient.

#### Accessory holders • First-class functionality

Three stainless steel accessory holders on both sides make it possible to install and use accessories very fast and easily.

#### **Central brake** • Activated with one single movement

Thanks to the centrally braked castors and the ergonomic chromium-plated brake bracket, the Olympia XLow bed simplifies patient treatments by the nursing

#### **Castors** • Superb maneuverability

High quality centrally braked design twin castors Ø 125 mm ensure a highly mobile

#### Mattress holders • Fixed mattress in each position

The backrest and footrest are equipped with 2 synthetic mattress holders.

\* option



## Comfort for everyone

The Olympia Hospital XLow bed is a perfect symbiosis of safety and comfort. A low bed position of  $\pm$  25 cm reduces the risk of falling, whereas the ergonomic working height of  $\pm$  80 cm guarantees efficient care.

**Sleeping position** 

the bed comfortably.

High position

The bed offers a sleeping position at an

the risk of falling considerably.

Getting in and out of the bed

extra low height of **25 cm**, which reduces

The bed stops automatically at **seat height** during a height adjustment. Thanks to this function, the patient can get in and out of

#### Height adjustment: 25 - 42 - 80 cm





### Care positions: Trend - CPR - Relax







#### (Reverse) Trendelenburg position

The reverse Trendelenburg position can be activated by means of the handset, whereas the Trendelenburg position can only be used after the nursing function has been activated by means of the magnetic key.

#### **CPR** position

After the nursing function has been activated, the CPR function brings the patient in a horizontal position at a height of **72 cm**. This position is ideal for both CPR and lifesaving care.

#### Seating or relax position

Both the backrest and knee rest are inclined simultaneously. A reverse Trendelenburg position is activated when a relax position is chosen on the Olympia XLow bed.



#### Easy to use a patient hoist •

The bed height can be easily adjusted, which enables the use of a patient hoist. The Olympia XLow bed is therefore ideal for the nursing staff to provide efficient daily care.

#### Selective locking system

The handset with selective locking system makes it possible to lock the bed functions selectively by means of a magnetic key.

#### The advantages are:

- 1 central handset to operate and lock the bed functions.
- Efficient activation by means of the handset, without straining your back.

#### Retracting backrest •

While inclining the backrest upwards, the backrest is automatically moved backwards. By doing so, pressure is relieved on the pelvis, whereas the bedsore risk is reduced considerably. Thanks to an integrated safety in the retracting backrest, there is no opening between the sideguards and the mattress platform during the inclination. This prevents the patient of getting stuck. The backrest of the Olympia Hospital XLow is also equipped with an inclination indicator.

001 Handset with selective locking system.	<b>004</b> A <b>reading lamp (*)</b> is a
002 An extendible drawer (*) can be used as a	OpenBus system, so that no
blanket rack.	005 Magnetic key
003 Under bed light (*)	006 Retracting backrest



The highest position of the bed is 80 cm.

The patient can be examined and treated

without straining your back.





control box




## • OpenBus intelligent

The OpenBus technology adds extra intelligence to the bed and its accessories. In the near future, OpenBus will also be able to register sensor data and pass this information to the nursing staff.

## • Under bed light

A discreet nightlight (\*) under the bed makes sure that the patient can find his way in a dim room, without having to disturb the other patients or nursing staff. The nightlight is activated by means of the handset with selective locking system.

amp (\*) is controlled and powered by the

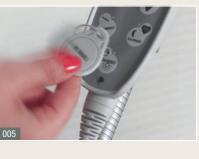



\* option

# Safety, mobility and flexibility galore

#### • Sideguards

The Trix sideguards (\*) consist of three siderails, offering maximum patient safety over the entire bed length. The sideguards can only be lowered by using your both hands. This safety system has been developed to prevent the sideguards from being lowered unintentionally.





007 The removable, perforated synthetic mattress platform sections (\*) ccan be cleaned very quickly and efficiently.

008 The Olympia Trix sideguards (\*) protect the patient over the entire bed length - even when the bed is extended - thanks to the unique telescopic system..

#### Central brake

A chromium-plated brake bracket activates the 4 castors simultaneously, and can be put in 3 different positions. The bed can be moved in each position, except when it has been put in the safety position (= lowest position).




4 braked castors



4 unbraked castors This position makes it possible to move the bed in all directions.

1 directional castor This castor makes it easier to move the bed in a straight line.

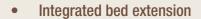

A hospital bed has to fulfil everybody's wishes, irrespective of the length of the patient. The integrated bed extension makes it possible to extend the bed with 20 cm.

When combined with the optional footrest extension (\*), a complete mattress platform functionality is guaranteed.



# Finishing and quality

#### HPL and solid laminate

A whole range of uni colours and wood patterns helps you to integrate the Olympia Hospital XLow bed perfectly in your institution. The Haelvoet laminate collection has a strong impact resistance and withstands scratches, light and damp. All laminate types are made in accordance with the norms DIN 53419 and EN 438.



N027 nover vénitien

N003 noyer du périgord



C132 chêne des marais




M091 merisier ambré

Both the bed panels and the bedside tables can be finished with these colours.



Other solutions for your interior decoration are possible on request.



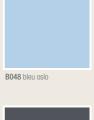

G076 gris acier



N056 noisetier naturel



T020 toraja wood

Standard Sanitized® treatment on all Polyrey laminates and solid laminates.


Partnership between Haelvoet and Polyrey, preferred supplier of solid laminate and HPL.

\* option



## Olympia Hospital XLow Summary

• Dimensions and performances

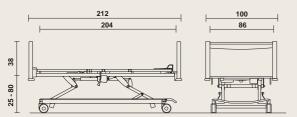

- length: 212 cm, width: 100 cm;
- height variation of the mattress platform:
   ± 25 cm ± 80 cm;
- time of the height adjustment: ± 25 sec.;
- Reverse Trendelenburg adjustment up to  $\pm$  16 °;
- clearance height: 135 mm (with a mattress platform height of 42 cm)
- safe working load: 230 kg;
- mattress platform dimensions: 204 x 86 cm;
- mattress dimensions: 195 x 85 cm, thickness: 12 to 15 cm.

#### • Electric protection class II

The bed is standard equipped with a double-insulated control box without earthing, equipotential, quick release and battery.




CE

Haelvoet makes sure that its excellent price-quality ratio is not at the expense of the environment. Our company considers durability – in all possible meanings – of paramount importance. All our products have a long lifespan and are delivered with a 10-year-guarantee on construction errors.

The Olympia XLow hospital bed is made in accordance with ISO 9001:2008 and complies with the European Medical Devices Directive 93/42/EEC and all other applicable European harmonised norms:

EN 60601-1 Medical electrical equipment - Part 1: General requirements for basic safety and essential performance.
EN 60601-1-2 Medical electrical equipment - Part 1-2: General requirements for safety and essential performance - Secondary norm: Electromagnetic compatibility - Requirements and tests.
EN 60601-2-52 Medical electrical equipment - Part 2-52: Particular requirements for basic safety and essential performance of medical beds.
EN ISO 14971 Application of risk management to medical devices.

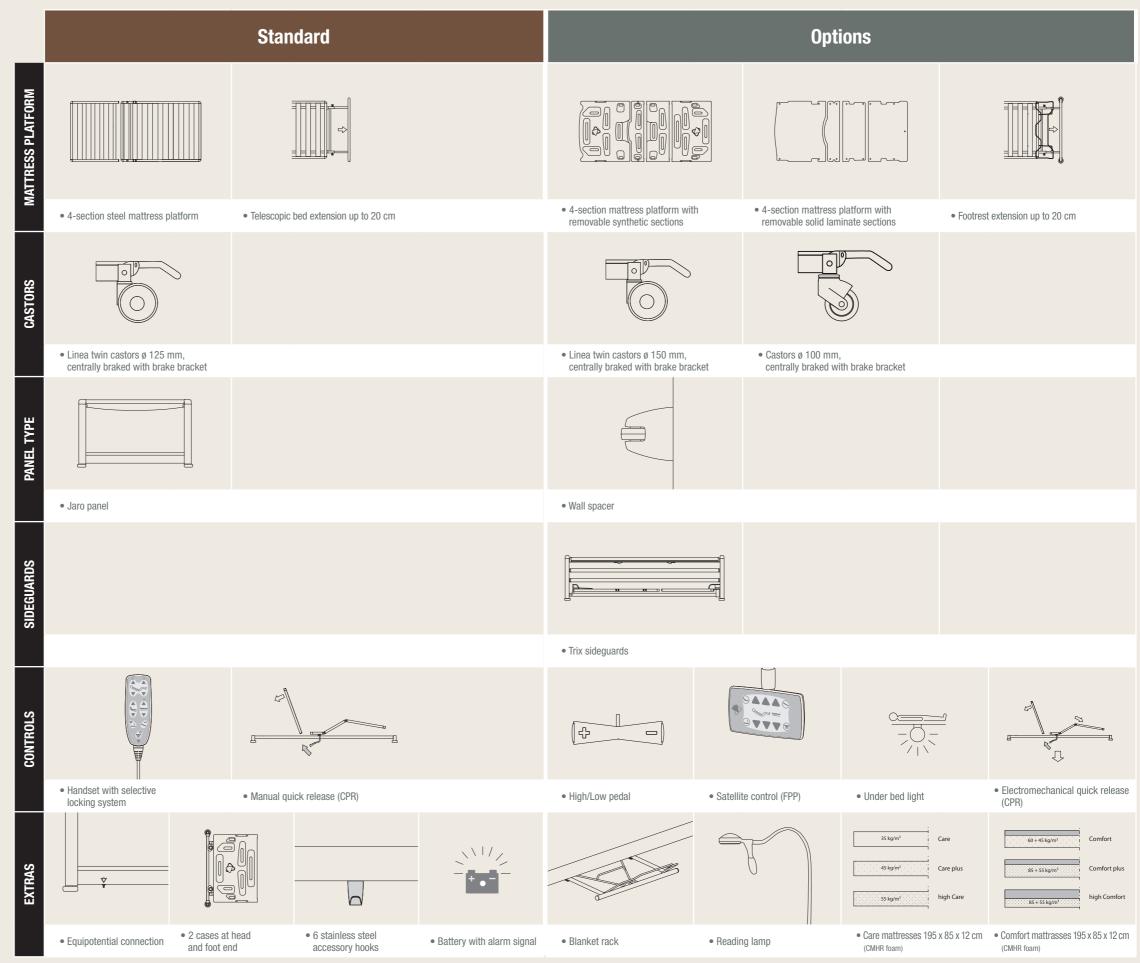

9

## Accessories

The accessories we offer can be harmoniously connected with the existing cases and accessory holders. Haelvoet strongly pays attention to this, so that the accessories offer you the same top quality as our hospital beds.



X-ray cassette holder

The X-ray cassette holder can be

easily attached to the backrest

platform section (only possible

with solid laminate sections).



